



**DXtwin**

## Introduction

### ▪ What is the DXtwin and why has it been developed?

The DXtwin is the STARVAC GROUP's very latest innovation in mechanical *palpate/roll* massage technology. Building on more than 40 years' experience in the field of beauty and health, we have combined the most advanced technological knowledge, which has been granted two new patents, to produce a revolutionary device designed for the treatment of cellulite and the redefining of body contours.

The palpate and roll massage technique has proved its effectiveness for a number of years. It helps to dislodge the fatty tissue (commonly known as cellulite) that accumulates in the subcutaneous layer.

The massage is performed with palpating movements over folds in the outer cutaneous surface, followed by a rolling action over the same area, hence the term palpate and roll.

The DXtwin is a device for professional depressotherapy (treatment by vacuum suction) which has various accessories and specific treatment protocols to help achieve three main objectives:

- **To assist lipolysis** : that is the melting of fat cells through the elimination of fibrous tissue and the reduction of adipose masses.
- **To re-generate blood and lymphatic circulation** : the draining and micro-stimulation actions of the treatment help to eliminate toxins and unblock subcutaneous tissue.
- **To redefine body contours** : the stimulation of elastin fibres leads to a smoother-looking and better-toned skin.
- **To glow the complexion** : by three combined actions like tonification, regeneration and drainage.


**Cellulite fat is resistant to calory-control and weight-loss, which makes the DXtwin a major and indispensable tool in slimming treatments.**

**With its advanced technology DXTWIN is the first device to offer a perfect mastery of connective tissues!**



## Description

- The DXtwin and its navigation screen

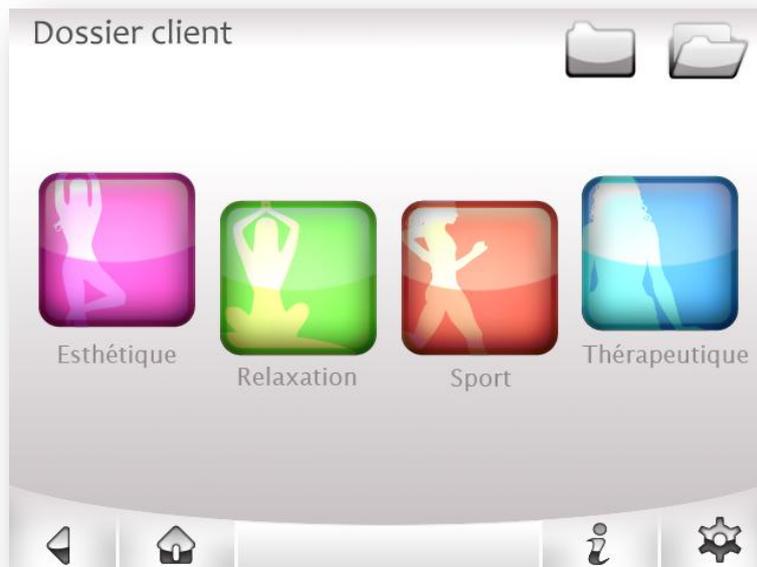

**Esthétique** : Stretch marks; Facial treatment; Scarring; Cellulite; Neckline; Treatments for men




**Relaxation** : Drainage; Heavy legs ; Relaxation



**Sport** : Tendinitis; Sprain; Muscle spasm; Muscle tear; Stiffness; Performance



**Thérapeutique** : Rheumatology; Sprain; Vascular problems; Scarring; Oedema  
Haematoma; Burns

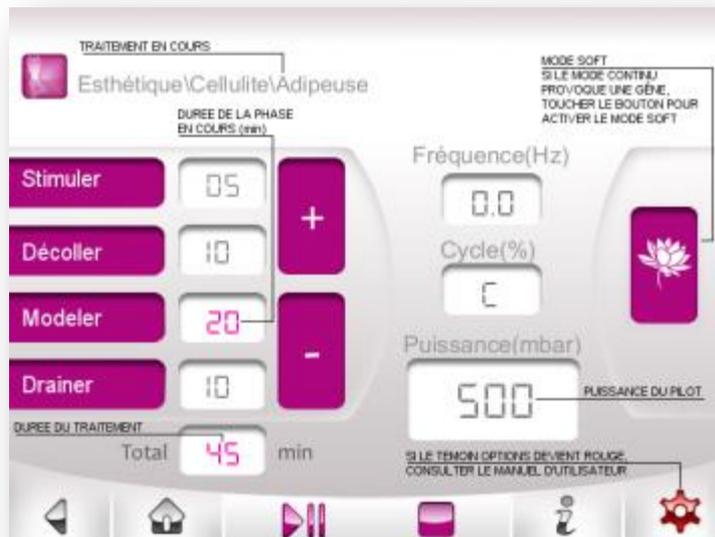

Once the appropriate programme is selected, the welcome screen appears showing the following information:

- The 4 types of treatment with the recommended duration for each protocol.
- The frequency: displays a faster or slower rate of the pulse mode.
- The treatment cycle: in pulse mode, this displays the percentage of strong aspiration.

The pulse mode alternates between a strong and a weaker aspiration, the latter allowing for the treatment of very painful cellulite.



This key enables the operator to move from using the EVOspheres to using the NEOcups.



The cotton-flower key shifts the action from continuous mode to pulse mode.



Returns to the preceding page.



Returns to the welcome page.



The help screen gives a summary of how to use the main functions of the touch screen.



The controls: Statistics; Modification of client files; Screen adjustment; Sound; Indication of the functioning of the device.

## THE DXTWIN AND ITS ACCESSORIES

### Pilot

The integrated system provides instant regulation of intensity at any moment during treatment, thereby allowing very precise adjustment to each treatment zone and to the reaction of the client. It also has a Start and Pause function for more rapid manipulation, as well as a visual indicator of the level of aspiration. Each accessory has its own individual safety connection. The command panel provides perfect regulation and control of treatment



### NEOCups


- ✓ A new more ergonomic shape makes for easy manipulation
- ✓ three sizes allow for different uses
- ✓ perfectly adaptable to the treatment area
- ✓ the NEOcups are made of transparent material, any skin reaction or sign of engorgement is instantly visible



## NEOlifts

- ✓ 3 suction heads dedicate to facial treatments
- ✓ usefull on small area (scars, stretch marks, wrinkles)
- ✓ different shapes according to their specific uses
- ✓ Pulsed mode optimized for the anti-aging treatment



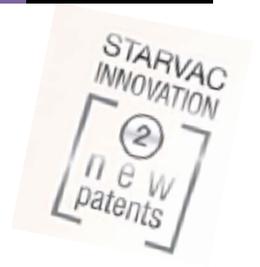

EVOspheres (A STARVAC innovation, granted a new patent)

Asymmetric motion a double fold of skin.

- ✓ allowing a deep elimination of fibrosis
- ✓ favouring a re-firming of the tissues
- ✓ adipocyte reduction
- ✓ ideal for treating flabby thighs and buttocks

Double asymmetrical mobilisation




HANDS **FREE**  
APPLICATION

EVO3D (A STARVAC innovation, granted a new patent)



## EVO<sup>3D</sup>

Action tridimensionnelle

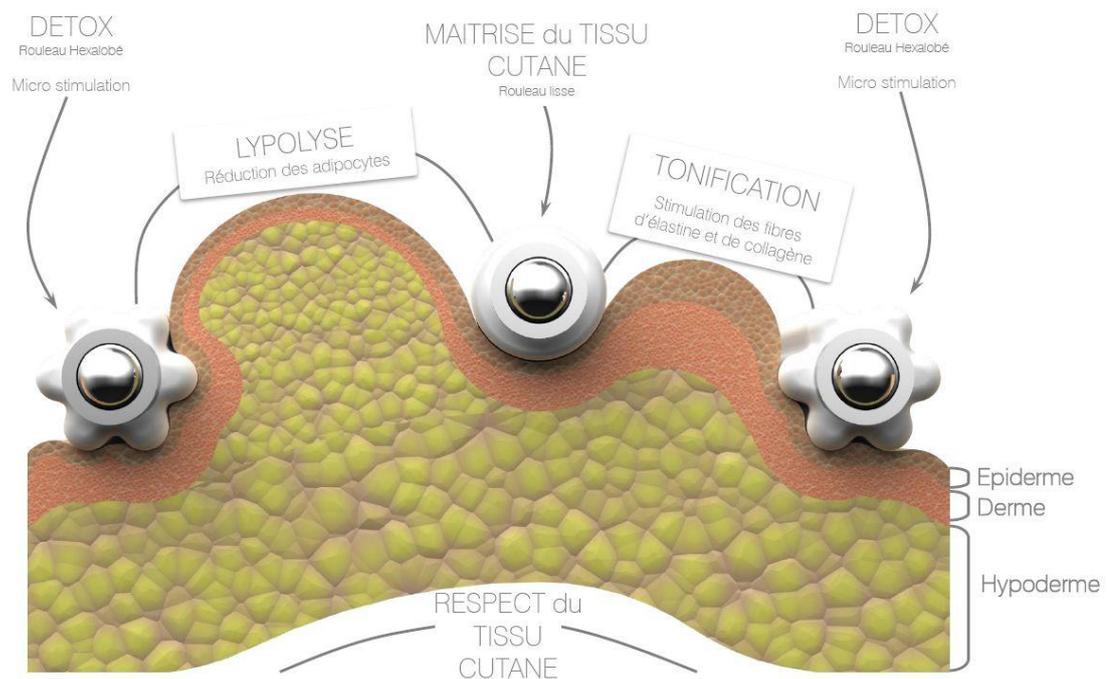

This is the first palpate and roll treatment head to use three rollers irregularly positioned at different widths and heights, such that two cutaneous folds of different thicknesses are formed. The three rollers are removable, making the EVO3D very easy to clean.

Thank's to this unique technologie and by the fact that the rollers are thin, the Dxtwin provide a very effective work using a low aspiration.

The EVO3D ergonomic shape softly applied bring a great sensation of Well-being.



## THE DXTWIN AND ITS SPECIALIZED COSMETIC RANGE

The DXTWIN technology makes it unique in the palpaté/roll market because it is the only one that enables the operator to use a cosmetic product at the same time as an accessory.

### Amincissante cream



*This is for use with all cellulite treatments as it contains active slimming ingredients, facilitates the movement of the EVO3D and is totally water-soluble, making cleaning simple and effective.*

**Lecithin** : encourages the circulation of fatty matter and prevents it from becoming trapped

**Caffeine** : has a diuretic effect and limits the depositing of fat cells, working also to micro-stimulate the tissues locally, hindering congestion and helping drainage

**Guarana** : is a plant containing a substance with a high concentration of caffeine (the highest in the world), increasing the metabolic and fat-burning rates



### Cryoactive drainage cream

*The Drainage Cream is used for treating heavy legs.*

**Menthol** : to give an intense refreshing and stimulating effect

**witch-hazel** } vegetable extracts : contain a vasoconstrictive property that decongests  
**ruscus** } and activates micro-circulation  
**ivy** }



### Regenerant serum

*It is used in the treatment of scars, stretch marks and wrinkles.*

**Organic silicium** : works on the regeneration of subcutaneous tissues

**Vitamin C** : powerful antioxidant, aids in the reconstituting of damaged and stretched tissues



### Restructuring gel

*The gel is used in facial treatments and especially for improving tonicity.*

**Royal jelly** : contains a large proportion of proteins, amino-acids and vitamins, giving it highly nutritive qualities that stimulate epidermic regeneration



### Purifying gel

*This gel is used in facial treatments, particularly in lymphatic drainage.*

**Tomato** : astringent and antiseptic

**Aloes** : softening and refreshing, toning and astringent properties. It is used to decongest the skin and has a lightening effect.

# Setting up the treatment

- **Bodycare expertise**

The Starvac Group has developed a bodycare expertise booklet specifically to go with the DXTwin. First of all this will help the beautician to establish good relations and an atmosphere of confidence with the client, but more particularly help in the choice of the most suitable treatment protocol to achieve the best results. Understanding life-style, determining the type of cellulite, diagnosing specific problems ... This is a work tool for the beautician, enhancing her professional skills.

**EXPERTISE CORPORELLE**

**Client Information:**

Nom: \_\_\_\_\_ Prénom: \_\_\_\_\_  
 Date de naissance: \_\_\_\_\_ Sexe:  F  M   
 Adresse: \_\_\_\_\_  
 Code Postal: \_\_\_\_\_ Ville: \_\_\_\_\_  
 Tél: \_\_\_\_\_ Portable: \_\_\_\_\_  
 Profession: \_\_\_\_\_  
 Avez-vous déjà effectué des soins corps:  OUI  NON   
 Si oui à quelle fréquence? \_\_\_\_\_

**ATTENTE DU CLIENT**

Gêlé  Étouffé  Jambes lourdes  Mège

**BILAN BEAUTÉ / SANTÉ**

**HYGIÈNE ALIMENTAIRE**

Appétit:  Faible  Normal  Excessif  
 Régulier  Irrégulier  Absent

Préférence pour les aliments:  
 Salés  Sucrés  Riches

Consommation journalière de fruits et légumes:  
 ..... / jour

Diabète:  
 Oui, QM Jour...  Non, QM Jour...  
 Café QM Jour...

Digestion:  
 Bonne  Lourde et / ou serrée après les repas

Fonctions intestinales:  
 Normale  Constipation  Diarrhées

**HYGIÈNE DE VIE**

Profil:  Sédentaire  Actif  
 Activité physique:  Oui  Non  
 Légalité: \_\_\_\_\_  
 Régularité: \_\_\_\_\_

Tobac:  
 Oui, nb...  Non  Antécédent

Produits corps utilisés à domicile: \_\_\_\_\_

Vous êtes stressée  
 Vous avez des problèmes de sommeil  
 Vous avez des problèmes de peau  
 Vous avez des problèmes de jambes lourdes/œdèmes  
 Vous faites de la rétention d'eau  
 Vous avez eu un bébé dans l'année

**Type de cellulite:**  Fibreuse  Adipose  Celluliteuse

**Stade cellulite:**  1  2  3

**Morphologie:**  Gynéko  Androïde  Mixte

**MESURE**  
 Taille: \_\_\_\_\_ m Poids: \_\_\_\_\_ kg (IMC: \_\_\_\_\_)

Mesure	Avant	Séances 1-10	Séances 11-20	Séances 21-30
Taille				
Hanches				
Poitrine				
Cuisse D				
Cuisse G				

Vêtement porté: \_\_\_\_\_ Taille: \_\_\_\_\_

**ÉVALUATION DE LA QUALITÉ DE LA PEAU - Note de 1 à 3**

Qualité de peau	Séances 1-10	Séances 11-20	Séances 21-30
Peau sensible			
Peu irritée			
Peu rouge			
Peu lisse			

**Actif sur les affections:**  
 Les zones à traiter  Les zones à éviter

## ▪ The choice of treatment protocol

After clinical trials, beauty and health professionals have determined the most suitable protocols for each case that is the number of sessions, the duration and frequency of the treatment.

Here are some examples of protocols to adopt:



### Stretch marks and scarring

24 sessions : 3 times a week



### Facials

16 sessions : twice a week



### Cellulite

12 sessions : twice a week

For heavy cellulite

18 sessions : twice a week



### Men's treatments

10 sessions : twice a week



### Neckline

12 sessions : 3 times a week



### Overall drainage and heavy legs

15 sessions : 3 times a week

## Marketing

- **New patents**

The DXtwin offers two newly patented innovations, the EVO3D and the EVOspheres. Communication based on a novelty, a completely new operating principle.



- **Institutional poster**

Communication based on a strong image of the brand. Well-known and reputable company with 40 years' experience in the beauty and medical fields, French-made, sold in 45 countries in all 5 continents, international patents



- **work tools**

Appointments diary, bodycare expertise booklet.




- Dual communication

Beauty care



Therapeutic care

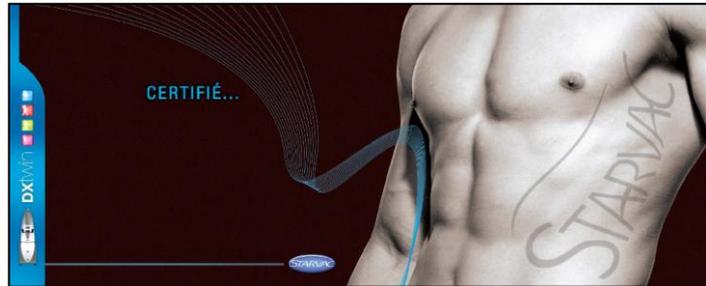

Advertisement in Les Nouvelles Esthétiques magazine

**STARVAC**

Palper Rouler dernière génération pour un traitement corps et visage

**INNOVATION STARVAC**  
2 nouveaux brevets

**EVOsphere**  
Double Mobilisation Asymétrique

**EVO3D**  
Action tridimensionnelle  
Maîtrise de l'élasticité du tissu conjonctif

**NEOcup**

**NEOlift**

- Défibrosage en profondeur
- Tonification et raffermissment
- Draine et élimine les toxines
- Micro-stimulation
- Efficacité accrue sur l'aspect peau d'orange.
- Stimule la lipolyse.

**DXtwin**

29 Rue Paule Raymondis • 31200 Toulouse  
+33 (0) 534 250 770 • [www.starvac-group.com](http://www.starvac-group.com)

**STARVAC**

Beauty & Medical Innovations

- PERSONALIZATION

The DXTWIN can be personalized in various ways: incorporation of a logo (institute, centre, brand, and sports team), choice of colour of the decorative fascia panel, choice of image.



6 Colour options for the fascia panel (Pastel blue, Lime tree green, Apple red, Fuschia pink, Silver, Metallic grey) and 2 theme (Nalio, Fluvia)

